

Keep this portion for your records

Live or dead ticks that have been removed from either humans or animals may be submitted. The result of the test is intended to provide you and your physician or veterinarian with important information to help determine risk of exposure and subsequent clinical treatment. Only 1 tick (from either a person or a pet) can be submitted per kit. The tick is tested using a sensitive DNA-based screening technique known as Polymerase Chain Reaction (PCR).

Regardless of test results, if you or your pet are experiencing any symptoms, consult your doctor or veterinarian.

Testing is provided by the Northeast Wildlife DNA Laboratory. Test turn-around is typically 1-3 business days after receipt of the tick. Test results will be sent to you via your preferred contact method. Please contact us at 1-855-812-0469 (toll free) for more information.


www.LymeAidKit.com

PLACE LABEL
HERE



Using this kit:

1. With the enclosed tick remover, approach the tick from the side with the tick remover hook until tick is secure.
2. Gently lift the hook and turn it to detach tick. The tick should detach after 2 or 3 rotations. Do not pull the tick. ROTATE until it detaches. See illustration.
3. Clean the area where the tick was attached with provided alcohol wipe.
4. Place tick into clear specimen bag, seal, and attach one of the 3 numbered labels.
5. Complete test payment method (see below). Cost of test is \$39.95.
6. Complete test submission form (see reverse). Place numbered labels where indicated. This number is your order number and will track the submitted tick through the testing and reporting process.

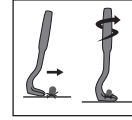

Payment Method: MONEY ORDER CREDIT CARD

Make money order payable to *Northeast Wildlife DNA Laboratory*, or fill out credit card information below if paying by credit card.

MASTERCARD VISA

CARD NUMBER _____ EXPIRATION DATE _____

NAME ON CARD _____ 3-DIGIT SECURITY CODE _____

INSTRUCTIONS

Services provided by: Northeast Wildlife DNA Lab
562 Independence Rd, Suite 114
East Stroudsburg, PA 18301
Telephone 1-855-812-0469



- ### Symptoms of Lyme Disease
- Upon infection, a bull's-eye shaped rash may appear within 4 to 20 days.
 - You may experience joint and muscle pain, nausea, fever, headache, chills, and fatigue.
- ### To prevent tick borne disease
- Avoid tick-infested areas
 - Wear light colored, tightly woven clothing
 - Use insect repellents
 - Tape openings in clothing
 - Conduct a body check upon returning from grassy or woody areas
 - Check pets for ticks
- ### Above all
- If bitten by a tick, use this kit to determine if the tick is a carrier of Lyme Disease.

Test Submission Form (Please print. Submit with the tick to be tested.)

NAME _____ DATE _____

ADDRESS _____

CITY _____ STATE _____ ZIP CODE _____

EMAIL _____ PHONE _____ FAX _____

PHYSICIAN OR VETERINARIAN NAME _____
PHYSICIAN OR VETERINARIAN PHONE/FAX _____

TICK EXTRACTED FROM: HUMAN ANIMAL - ANIMAL TYPE: _____

LYME-AID TESTING KIT PURCHASED AT: _____

CHECK PREFERRED RESPONSE METHOD(S):
 EMAIL PHONE FAX MAIL

PLACE LABEL
HERE